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
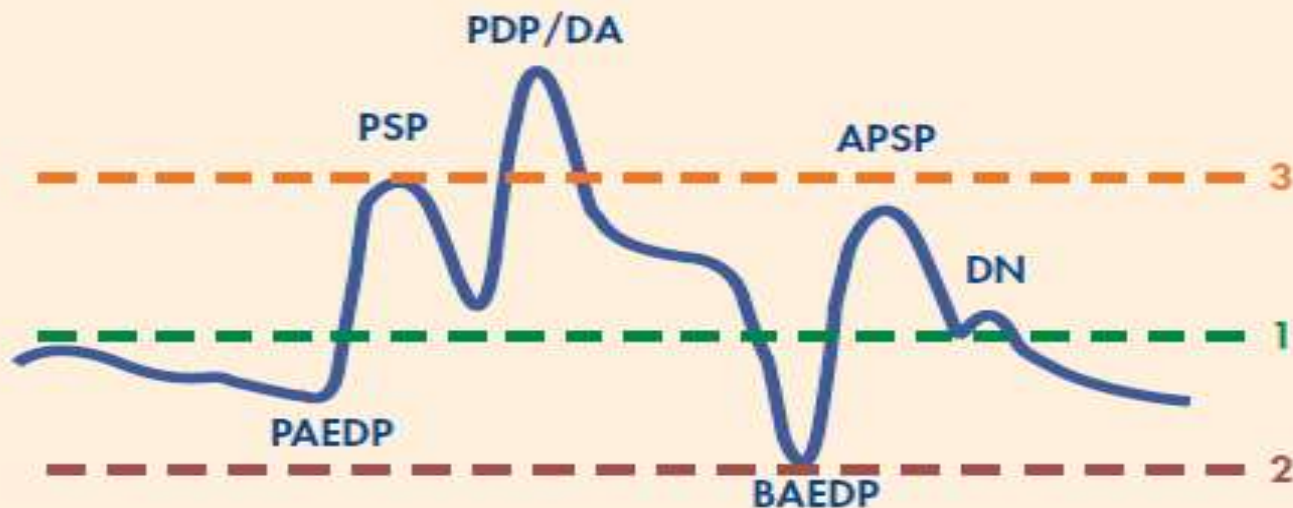
Assisted Arterial Waveform (1:2)

Abbreviation	Definition
PAEDP	Patient Aortic End Diastolic Pressure
PSP	Peak Systolic Pressure
PDP/DA	Peak Diastolic Pressure/Diastolic Augmentation
BAEDP	Balloon Aortic End Diastolic Pressure
APSP	Assisted Peak Systolic Pressure (Systole after IAB deflation)
DN	Dicrotic Notch

Early Inflation

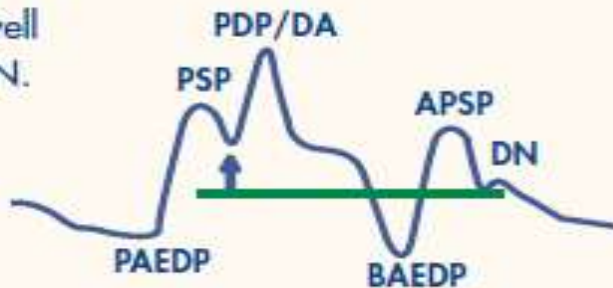
IAB is inflated well before actual DN.
(aortic valve closure.)
Violates **Rule 1** for inflation.

Result: Premature closure of aortic valve.
Reduces stroke volume/CO.
Increase in LVED volume.

Late Inflation

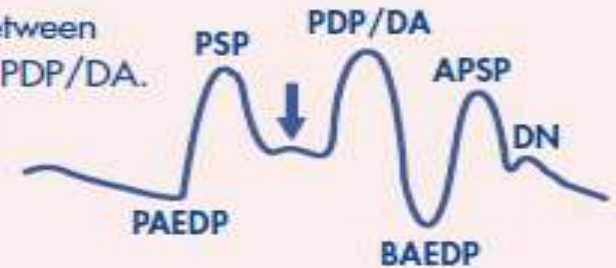
DN is visible between points PSP and PDP/DA.
Violates **Rule 1** for inflation.

Result: PDP/DA less than optimum.
Decreased perfusion pressure and volume to coronary arteries.

Early Deflation

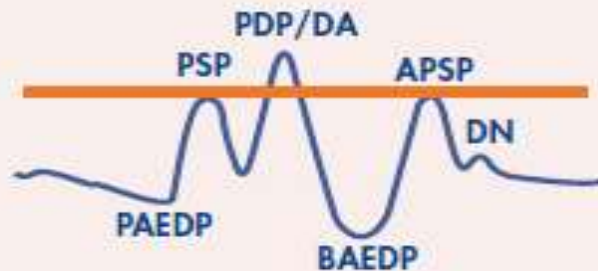APSP = PSP
Violates **Rule 3** for deflation.
May see "U" shape at BAEDP.

Result: No afterload reduction.

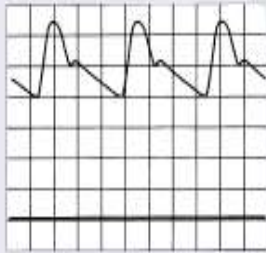
Late Deflation

BAEDP > PAEDP
Violates **Rule 2** for deflation.


Result: Increased workload of left ventricle.
Increased MVO₂.
Decreased CO.

Purge Failure

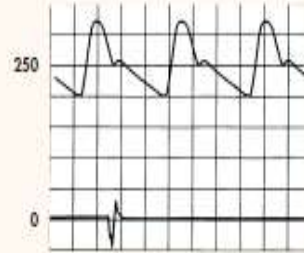
Check for: • Loss of trigger

Purge Failure

Check for: • Leaks
• Low helium

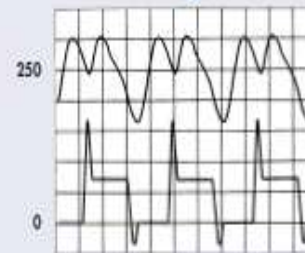
Reduced Augmentation

Low Plateau Pressure

Check for: • Volume setting too low
• Balloon too small for patient
• Balloon too low in aorta
• Low systemic vascular resistance

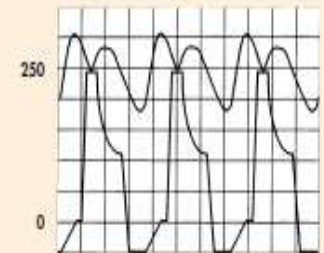Poor Augmentation

Wide Inflation and/or Deflation Artifact

Check for: • Proximal portion of IAB in sheath
• Suture too tight around catheter
• Partial obstruction
• Partial kink
• Slow catheter or He shuttle speed
• Very tortuous vessels

Wide deflation artifact may cause
Possible Helium Loss Alarm in 1:1 assist.

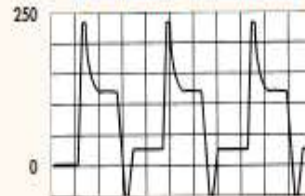
24-Hour Intra-Aortic
Balloon Product Hotline:
800-447-IABP

Worldwide: 617-389-8628

ARROW
INTERNATIONAL

2400 Bernville Road
Reading, PA 19605
610-378-0131
800-233-3187


Baseline Elevated

High Baseline

Check for: • Kinked catheter
• Partially wrapped balloon
• IAB in sheath
• IAB too low in aorta
• IAB too large
• Overfill

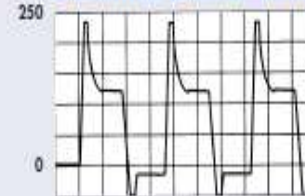
Baseline Below Zero

Possible Helium Loss

Check for: • Blood in catheter tubing
• Possible leak in connections or tubing
• Kinked catheter
• Ectopic beats

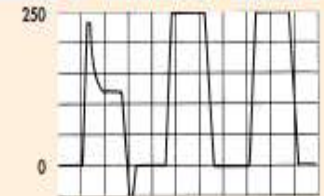Squared or Rounded Plateau Pressure

High Pressure

Check for: • Partially wrapped balloon
• Kink in catheter or tubing
• Balloon too large for aorta
• Balloon position too high or too low
• Balloon in sheath

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INDICATIONS	CONTRAINDICATIONS
Left ventricular failure or cardiogenic Shock a. Myocardial Infarction (MI) b. Myocarditis c. Cardiomyopathy d. Severe myocardial contusion e. Septic Shock f. Drug induced	<ul style="list-style-type: none"> Severe Aortic valvular insufficiency
Mechanical Complications of Acute MI	<ul style="list-style-type: none"> Aortic dissection
Unstable Angina - Myocardial Infarction - Ventricular Irritability	<ul style="list-style-type: none"> Severe peripheral vascular disease
Unstable Angina refractory to medical therapy	<ul style="list-style-type: none"> Irreversible Brain Damage
Support for High risk PTCA - Patients	
Failed PTCA	
Thrombolytic Therapy of Acute MI	
Failure to wean from Cardiopulmonary Bypass	
Low - Output Syndrome - Stabilization of High - Risk Patients - Undergoing General Anesthesia - Bridge to Transplant - Ischemic Myocardium	

Table 4: Complications of IABP counterpulsation

VASCULAR	BALLOON related	MISCELLANEOUS
Arterial injury (perforation, dissection)	Perforation	Hemorrhage
Aortic perforation Aortic dissection	Tear	Infection
Femoral artery thrombosis	Rupture	Entrapment
Peripheral embolization	Incorrect positioning	
Femoral vein cannulation	Gas embolization	
Limb ischemia		
Visceral ischemia		

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